



# USER MANUAL

**Drain bag holder**

**MARTS 2023**

## Table of content

Important information .....	3
Intended use .....	3
Storage .....	3
Daily operation .....	4
Warnings .....	4
Environmentally friendly disposal .....	4
Technical datasheet .....	5
Technical datasheet .....	6

## **Important information**

Before using the Drain bag holder read these instructions closely.

## **Intended use**

Henrotek drain bag holder are only intended to be used for drain bag that can uptrain up to 2000 ml fluid and on a bedposts.

Has a fastening function on the urine bag hanger for the urine bag hose, solid and safe design that holds the hose while insuring a safe passage for fluid.

Drain bag holder is design and intended for holding drain bags with up to 2000 ml on the bedposts. Drain bag holders can support different kinds of urine bags.

The hooks on the blue bag holders are rounded in order to fit hospital beds with round bedposts.

The hooks on the green bag holders are more in a square shape in order to fit hospital beds with square bedposts.

The drain bag holders have been tested with 2500 ml of liquid for 24 hours without any reduction in quality. These tests were conducted by the quality control of an international group specialized in the health sector.

The drain bag holders consist of a hook and a hanger that are easily assembled.

## **Storage**

Store dry, at room temperature and out of direct sunlight.

## **Instruction for use**

Content: 1 Hook and 1 Hanger

1. Unpack the hook and hanger.
2. Ensemble the hook to the hanger.
3. The Drain bag holder are now ready for use.

## **Precautions**

Do not bend any of the parts on the hook or hanger.

Do not put any of the parts over candle light or fire.

## **Daily operation**

Drain bag holders are autoclavable up to 127° C degrees.

## **Warnings**

If there appears to be any sharp edges that may damage the drain bag, do not use and discard of the drain bag as mentioned in this user manual.

## **Environmentally friendly disposal**

Please dispose the cup in an environmentally friendly neutral way.



Our drain bag holder are made of recyclable material which should be recycled. Please therefore dispose the old drain bag holder via suitable collection systems.

## Technical datasheet



<b>Material Cup</b>	Polyamide 6
<b>Weight</b>	28 G
<b>Height</b>	11 cm
<b>Width</b>	18 cm
<b>Marked</b>	CE
<b>Max load Milliliters</b>	2000 ml
<b>Dishwasher</b>	127° C degrees autoclavable
<b>Basic UDI No.</b>	57048317C8
<b>GTIN14 Drain Bag Blue Hanger</b>	05704831128203
<b>GTIN14 Drain Bag Blue Hook</b>	05704831128197
<b>Standards and Regulations</b>	Regulation (EU) 2017/745 of 5 April 2017

## Technical datasheet



<b>Material Cup</b>	Polyamide 6
<b>Weight</b>	30,5 G
<b>Height</b>	10 cm Squared hook
<b>Width</b>	18 cm
<b>Marked</b>	CE
<b>Max load Milliliters</b>	2000 ml
<b>Dishwasher</b>	127° C degrees autoclavable
<b>Basic UDI No.</b>	57048317C8
<b>UDI-DI Green Drain Bag Hanger</b>	05704831128173
<b>UDI-DI Green Drain Bag Hook</b>	05704831128180
<b>Standards and Regulations</b>	Regulation (EU) 2017/745 of 5 April 2017