



InterDry® – tekstil indeholdende et antimikrobielt sølvkomplex til behandling af bakterie- og svampeinfektioner i hudfolder


Guide til påsætning af InterDry i hudfolder

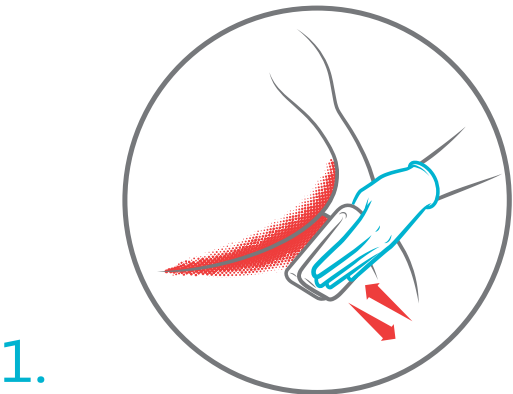


1.
Vask området forsigtigt.
Dup huden tør.

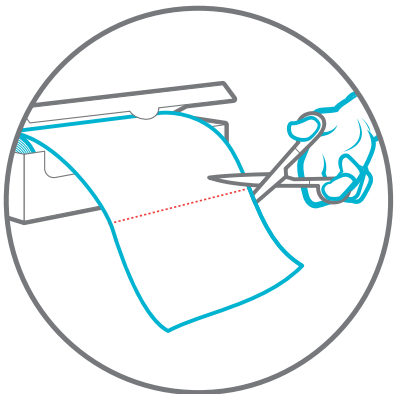


2.
Klip et stykke tekstil med en
ren saks,

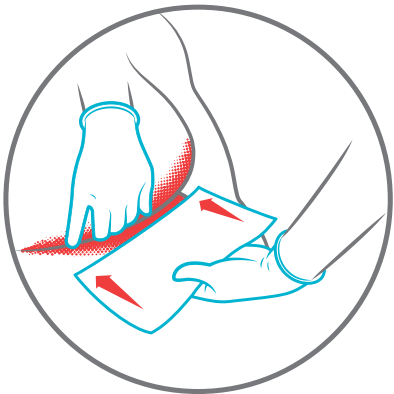

3.
Læg et enkelt lag tekstil helt ind i
bunden af hudfolden.

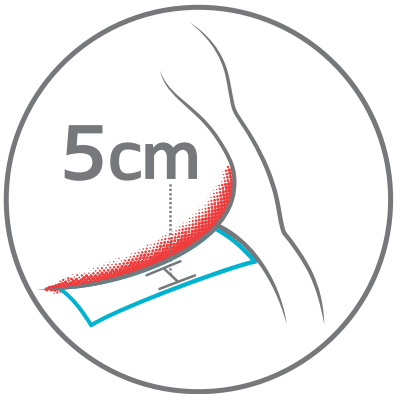


4.
Efterlad mindst 5 cm tekstil uden
for hudfolden.

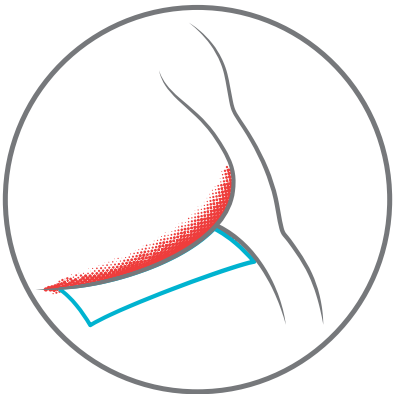

5.
Fiksér tekstilet i hudfolden, eller holdt
fast under tøj, f.eks BH.



6.
Brug af pudder, salve eller cremer kan
hindre, at InterDry virker efter hensigten.

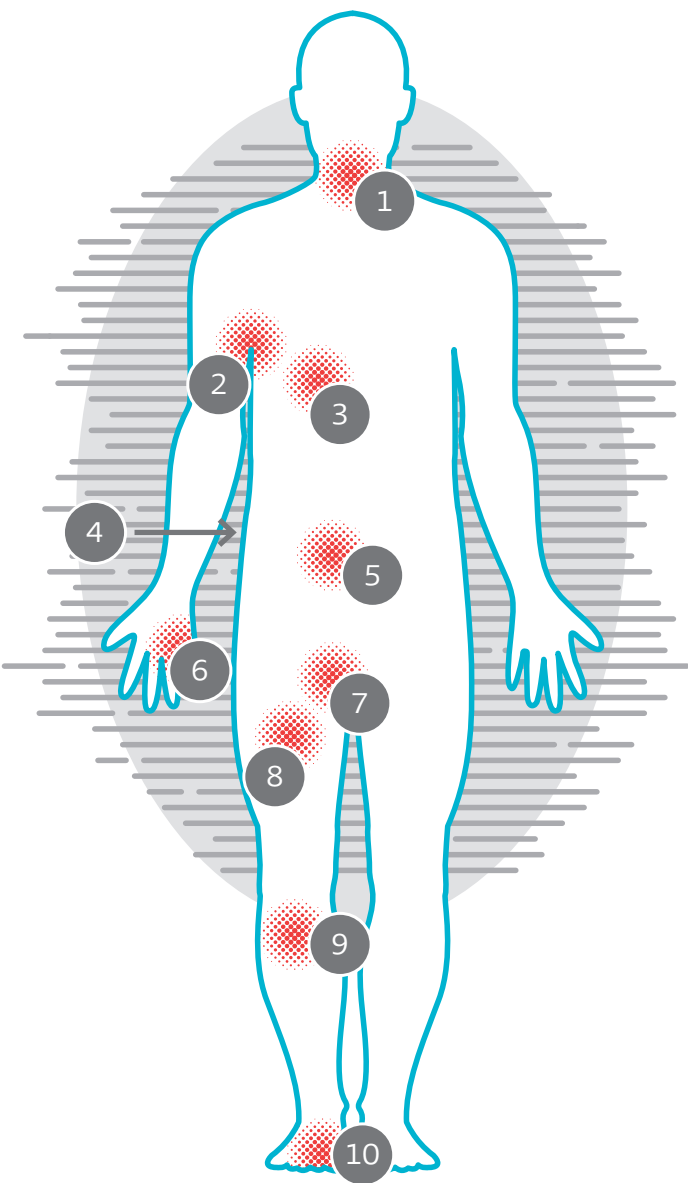

Almindelige områder med intertrigo:¹⁻³

1. Nakke
2. Armhule
3. Under brystet
4. Ryg
5. Mave
6. Mellem fingre
7. Lyske
8. Lår
9. Underben
10. Mellem tæer

Se brugervejledningen for en mere
detaljeret vejledning i brug af InterDry.

InterDry		
Varenummer	Størrelse	Enhed
67918	25 cm x 366 cm	1 rulle
67919	25 cm x 91 cm	10 stk/pr. æske



Referencer:
1. R. Gary Sibbald et al. A Practical Approach to the Prevention and Management of Intertrigo, or Moisture-associated Skin Damage, due to Perspiration: Expert Consensus on Best Practice. Wound Care Canada, Volume 11, Number 2 - Fall 2013 Accessible at www.woundcarecanada.ca/supplements.
2. Janniger CK, Schwartz RA, Szepletowski JC, Reich A: Intertrigo and common secondary skin infections. Am Fam Physician 2005, 72(5):833-838.
3. Gray (2011). Moisture-Associated Skin Damage Overview and Pathophysiology.