(en)

Indications: For use on adhesives, dressings and appliances on skin.
DESCRIPTION OF DEVICE & CONTENTS: ESENTA™ medical adhesive remover wipes sting free
INTENDED USE: Removal of adhesive residues from skin
INSTRUCTION FOR USE:

- Tear sachet to open and unfold wipe
- Rub the wipe over adhesive residues to remove

INFORMATION: Dries in a few seconds. Create a clean surface without affecting the adhesion of the next appliance or dressing.
CAUTION: This device is for single-use only and should not be re-used. Re-use may lead to increased risk of infection and cross contamination. Physical properties of the device may no longer be optimal for intended use. After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations.
If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

- Avoid direct use on wound beds.
- Avoid use on delicate or sensitive areas of skin (eyes, mouth, ears, nose, vagina).
- If in doubt, patch test a small area prior to use.
- Should skin irritation occur, discontinue use and consult a healthcare professional.
- For external use only.

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