

GRIPPER PLUS®

Safety Huber Needle



Access a port

Prepare the site according to facility protocol. Access a port with the **GRIPPER PLUS®** safety needle.



Remove clip

Remove the clip by sliding it towards the end of the needle arm and lifting.



De-access with two fingers on base

To de-access the port, approach the **GRIPPER PLUS®** safety needle from behind. Place fingers on the base to stabilise it.



Lift safety arm

With the other hand, place a finger on the tip of the safety arm. Lift the safety arm straight back. Notice that the needle comes out perfectly straight.




“Click” needle into locked position

Continue lifting until the needle “clicks” into the locked position. The **GRIPPER PLUS®** safety needle is now ready for disposal into a sharps container.

Now Available for Power Injection

GRIPPER PLUS® POWER P.A.C. Safety Needle for Power Injection

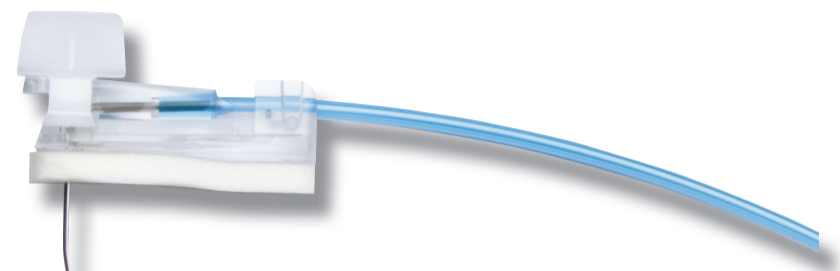

- Blue tubing provides easy identification for power
- Printed clamps show power specifications



ORDER INFORMATION

GRIPPER PLUS® POWER P.A.C. Needle Sizes

Available in non Y-site and Luer-activated needless Y-site (NAC)

Gauge	Length		Product Code	
	mm	inches	non Y-site	NAC
19	19	¾	21-3368-24	21-3468-24
19	25	1	21-3364-24	21-3469-24
19	32	1 ¼	21-3365-24	21-3470-24
20	16	⅝	21-3369-24	21-3464-24
20	19	¾	21-3367-24	21-3465-24
20	25	1	21-3362-24	21-3466-24
20	32	1 ¼	21-3363-24	21-3467-24

Do not power inject contrast media unless a **PORT-A-CATH® POWER P.A.C.** system has been verified

ORDER INFORMATION

GRIPPER PLUS® Needle Sizes

Available in non- Y-site split septum and Luer-activated -site

Gauge	Length		Product Code		
	mm	inches	non Y-site	Y-site	NAC
19	19	¾	21-2768-24	21-2868-24	21-2968-24
19	25	1	21-2764-24	21-2869-24	21-2969-24
19	32	1 ¼	21-2765-24	21-2870-24	21-2970-24
20	16	⅝	21-2769-24	21-2864-24	21-2964-24
20	19	¾	21-2767-24	21-2865-24	21-2965-24
20	25	1	21-2762-24	21-2866-24	21-2966-24
20	32	1 ¼	21-2763-24	21-2867-24	21-2967-24
22	16	⅝	21-2770-24	21-2860-24	21-2960-24
22	19	¾	21-2766-24	21-2861-24	21-2961-24
22	25	1	21-2760-24	21-2862-24	21-2962-24
22	32	1 ¼	21-2761-24	21-2863-24	21-2963-24

THE DETAILS GIVEN IN THIS DOCUMENT ARE CORRECT AT THE TIME OF GOING TO PRESS. THE COMPANY RESERVES THE RIGHT TO IMPROVE THE EQUIPMENT SHOWN.

Smiths Medical International Ltd
Military Road, Hythe, Kent CT21 6JL, U.K.
Tel: +44 (0)1303 260551
Fax: +44 (0)1303 266761
www.smiths-medical.com

Smiths Medical, part of the global technology business Smiths Group


GRIPPER PLUS, Deltec and the Smiths design mark are trademarks of the Smiths Medical family of companies. The symbol © indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries. Literature No. LIT/VA2923 (sheet 1) © 2008 Smiths Medical family of companies. All rights reserved.

Deltec™

GRIPPER® Micro

Blunt Cannula, Non-Coring Safety Needle

Insertion

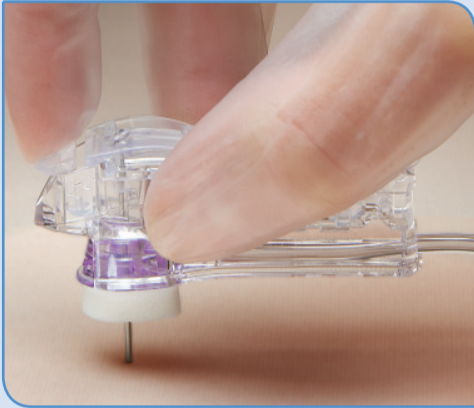

Access a port

Prepare the site according to facility protocol. Holding on to the inserter as shown, insert the needle into the port.



Remove the needle

- From the back of the inserter, place fingers on each side of the inserter's base to stabilise it.
- With the other hand, place a finger on the tip of the inserter's safety arm.
- **Press the tab in** and lift the safety arm straight back until the needle **CLICKS** into the locked position.

✓ **The sharp is now removed.**



Dispose in sharps container

- Dispose of the inserter in a sharps container.
- Apply a semi-permeable dressing over the infusion site, ensuring that a minimum 4cm area surrounding the site is covered.

Removal



Remove infusion site

Place fingers on each side of the infusion site. Stabilise the port with the other hand. Lift the infusion site straight up and discard per facility protocol.

ORDER INFORMATION

GRIPPER® Micro Needle Sizes

Available with a non Y-site and Luer-activated needleless Y-site (NAC)

Gauge	Length		Product Code	
	mm	inches	non Y-site	NAC
19	19	¾	21-3260-24	21-3280-24
19	25	1	21-3261-24	21-3281-24
20	19	¾	21-3256-24	21-3276-24
20	25	1	21-3257-24	21-3277-24
22	19	¾	21-3251-24	21-3271-24
22	25	1	21-3252-24	21-3272-24

THE DETAILS GIVEN IN THIS DOCUMENT ARE CORRECT AT THE TIME OF GOING TO PRESS. THE COMPANY RESERVES THE RIGHT TO IMPROVE THE EQUIPMENT SHOWN.

Smiths Medical International Ltd
Military Road, Hythe, Kent CT21 6JL, U.K.
Tel: +44 (0)1303 260551
Fax: +44 (0)1303 266761

Smiths Medical, part of the global technology business Smiths Group

GRIPPER PLUS, Deltec and the Smiths design mark are trademarks of the Smiths Medical family of companies. The symbol © indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries.
Literature No. LIT/VA2923 (sheet 2) © 2008 Smiths Medical family of companies. All rights reserved.

Deltec™